

The Sitting and Rising Bed

Special Nursing Beds





mobilia Sitting and Rising Bed

Mobility for patients and easier work for nurses

The mobilia sitting and rising bed in its different versions casa, cura and integra enables the patient to get up from the bed again on his own or with very little support.

Turning lying surface – new mobility

The mobilia nursing bed series is suitable for care at home or in an institution. The effects of many illnesses include physical mobility limitations. Such limitations often affect the patient's ability to move. Turning movements are often impossible, sitting up in bed is difficult or even impossible and will considerably limit quality of life.

The mobilia sitting and rising bed is used for many different disabilities and illnesses, e.g. in case of movement impairment, too-low muscle strength, inability to leave the bed, support at limited stress-capacity of the lumbar spine or joints when sitting due to cardiac insufficiency or for mobilisation, etc.. From the mobilia nursing bed, with all standard functions of a regular nursing bed and an additional sitting position as well as the optional rising aid, patients can better see their environment.

The patient's mobility and transfer are made much easier and perceived as pleasant by the patient. The mobilia sitting and rising beds also give the patient new independence and enable him to rise on his own again. For family performing nursing tasks, the nursing environment and nursing staff, the mobilia series makes things much easier. It prevents occupational diseases in nursing staff, supports family members in nursing and improves the patient's quality of life.

Easy mobility – with the rising bed from the mobilia nursing bed series.

Comfort

Comfortable lying feeling from

- · 2-part mattress
- · Rising aid with triangle grip
- · Special elastic cover (everything included)
- · Arm rests available optionally



Function

- Extreme facilitating of rising and fully motor-powered turning lying surface
- · Electrical adjustment of the back, thigh and lower leg rests as well as lifting and rising aid
- · Lying surface rotation to either side optional
- · Mobilia casa: single wheel braking
- · Mobilia cura: single or 2-wheel central braking
- · Ground clearance for lifter
- · Max. patient weight 100 kg

Design

· Comfortable and beautifully shaped design of the head and foot parts

Areas of use

- Nursing at home, in senior's residences and nursing homes, hospices, rehabilitation facilities and retire ment homes, hospices, rehabilitation facilities and retirement homes
- The mobilia integra bed insert can be used in personal applications as well as for special uses, e.g. in hotels or cruise ships (for present double or single beds)

Other wood patterns available on request
Optionally with steel grid lying surface
Metal coating in different RAL colours
Lying surface widths 90 cm or 100 cm
bed extension to 220 cm possible



Functional Sequence



1. Lying position



2. Head and foot part adjustment



3. Turn of the LS by 45°



4. Turn of the LS by 90°



5. Sitting position



6. Rising position





mobilia cura Shear Lifting Bed

The stable transforming artist

mobilia cura (with its versions mobilia cura, cura e and cura e plus) is one of the two basic versions of our mobilia range. In its basic construction, it is based on a shear-lifting system that particularly represents extreme stability. It also entices with a clear, light-weight design and integrates into any present apartment furnishings.

- · Lying surface rotation to either side optional
- · Electrically adjustable back, thigh and lower legs rests and height adjustment plus rising aid
- · Ground clearance for lifter
- · Lying and sitting height smoothly adjustable from 40 80 cm (mobilia cura e plus 48 - 80 cm)

Equipment versions



mobilia cura e plus

- · Fully electrical lying surface adjustment with rising aid
- · Fully electrical turning and rising function



mobilia cura e

- · Fully electrical lying surface adjustment
- · Fully elctrical turning function of the LS



mobilia cura

- · Fully electrical lying surface adjustment
- · Manual turning function of the LS







1

mobilia casa Front Lifting Bed

The elegant transforming artist

mobilia casa (with its versions mobilia casa, casa e and casa e plus) is the second basic version of our mobilia range. In this version, the lifting system is integrated into the foot and head parts. This elegant solution offers perfect ground clearance for moving lifter systems under the bed and facilitates cleaning below the bed as well.

- \cdot Lying surface rotation to either side optional
- · Electrically adjustable back, thigh and lower leg rests, height adjustment and rising aid
- · Ground clearance for lifter
- · Lying and sitting height smoothly adjustable from 40 80 cm (mobilia cura e plus 48 80 cm)

Equipment versions



mobilia casa e plus

- \cdot Fully electrical lying surface adjustment with rising aid
- Fully electrical turning and rising function



mobilia casa e

- · Fully electrical lying surface adjustment
- \cdot Fully electrical turning function of the LS



mobilia casa

- · Fully electrical lying surface adjustment
- · Manual turning function of the LS









mobilia integra Bed Insert

The flexible retrofitting solution for present single or double beds

Physical limitations often happen quite suddenly. A solution must be found quickly to relieve the patient and nurses. For this case, Mühle Müller Pflegebetten offers a mobilia bed version that takes care of such concerns – mobilia integra with all functions of the mobilia system as a pure functional element without bed frame. The mobilia integra can be integrated into any present single or double bed and thus is the perfect solution for anyone who wants to stay in his or her usual home environment.





mobilia integra e

with rising aid

function

· Fully electrical lying surface adjustment

mobilia integra e plus

· Fully electrical lying surface adjustment

• Fully electrical turning and rising

· Fully elctrical turning function of the LS



mobilia integra

- · Fully electrical lying surface adjustment
- · Manual turning function of the LS











mobilia Models

Your individual mobilia in a comfortable wood design







hamilton

All mobilia models are optionally available in many high-quality wood and design versions – special wishes will be met as well.

This way, you can design any mobilia according to your personal ideas and adjust it to your personal living situation.





Werner Mühle Metallverarbeitung GmbH & Co. KG Boschstraße 6 35799 Merenberg Germany

Phone: +49 (0) 6471 - 51 65 00 Fax: +49 (0) 6471 - 51 650 22

E-Mail: info@muehle-mueller.de Web: www.muehle-mueller.de